

# Stalham Sani/Shower Chairs

## General step by step guide to dismantling and decontamination

There are obviously many varied cleaning methods (steam, autoclave etc.) & numerous decontamination solutions (sprays, wipes etc.) adopted throughout the NHS but please follow the general guidelines below in conjunction with your own Trusts recommendations (additional notes may be added where required)

1



Put on personal protective equipment

Notes:-

2



Collect cleaning solution, prepare if necessary

Notes:-

3



Fully dismantle unit by removing over-seat, aperture, bedpan rack, footrest and back pad if fitted

Notes:-

4



Thoroughly clean and dry all removable parts

Notes:-

5



Thoroughly clean and dry frame from above and below including arms

Notes:-

6



Reassemble unit and write date and initials on decontamination tag

Notes:-

7



At least weekly turn the commode over and thoroughly clean and dry the castors whilst checking the whole unit for any signs of damage or wear

Notes:-

If you have any questions please contact our Sales Department on **01535 272957**